

Post this Side in Your Sterilization Room.

How to use Handpiece Lubricant



Lube here



Lubrication Tools



1

Spray

Using the correct adapter, spray MK-dent spray into the drive line or the hole. Spray for approx. 2 seconds - Count 1..... 2.....



Once a week the spindle of the handpiece has to be lubricated

2

Run 5 to 10 seconds

Run the handpiece to expel out excess lubricant. This should be done at the control unit in the operator or at the control unit in the operator or at a purge station in the sterilization room
Count 1..... 2 3..... 4..... 5.....



3

Wipe with alcohol to remove excess lubricant

Wipe down the exterior of the handpiece with a 2x2 lightly dampened with alcohol. Use pure rubbing alcohol only, many over-the-counter "alcohol wipes" have other solutions added to them, this can cause spotting of the exterior and severe damage to the internal parts of the handpiece.Count 1..... 2...



4

Bag and Autoclave

Sterilize handpiece as per manufacturer's instructions. Always allow the handpiece to go through the "Dry Cycle" on your sterilizer. Note: We recommend using sterilization bags with one side paper and side plastic.



5

Let Cool

Remove the handpiece from the sterilizer and let them cool to room temperature. Running "Hot" handpieces will severely reduce the life of the turbine.

- Don't Re-Lubricate
- With MK-dent Spray You Lubricate Only Once!



Caution:

- *Never Wipe or Spray With Surface Disinfectants
- *Never Submerge In Water * Never Place Handpiece Into The Ultrasonic Cleaner